

# baremedical<sup>®</sup>

## Catheter Strap

Baremedical catheter straps are made from surgical stretch material that is gentle on the skin, limiting the risk of irritation and potential infections due to tears from tape. By keeping the tubing secured, the catheter strap reduces the chance of catheter pull-through. Each strap contains easy to follow instructions for use.

### Features & Benefits

- Hook and loop closure
- Dual locking tabs
- One size fits most

### Instructions For Use

- Position leg band high on patient's thigh with locking tabs toward the inside front of the leg. Tighten leg band and secure with the hook and loop tab. A proper fit allows two fingers to fit snugly under the band.
- Place catheter over the middle of the locking tabs where the locking tabs are stitched to the leg band.
- Take the narrow locking tab over the catheter and insert through the square cutout on the wider locking tab. Fasten to the leg band
- Take the wider locking tab and fasten to the leg band in the opposite direction.



### PRODUCT INFORMATION

Material	Polyester, Nylon and Spandex
Sterility	Non-Sterile
Latex Status	Latex Free

### ORDERING DETAILS

SIZE/STYLE	60cm
CH2 CODE	2556141
UNIT OF MEASURE	Pack of 10