

# baremedical<sup>®</sup>

## Urine Leg Bags

Baremedical urine leg bags are available in both 500ml and 750ml with sterile and non-sterile options. Leg bags are generally strapped to the thigh using a strap or securement device which makes it more comfortable for patients to go out and about.

### Features & Benefits

- Integrated anti-reflux device
- Not visible when worn under pants or long skirt
- Made from sturdy plastic material to prevent leakage.
- Flip tap for easy emptying with low risk of spillage
- Perforated tear line on the non-sterile packs for ease of opening and accessing the loose bags inside.

### Instructions For Use

- Ensure that the Flip-Eze tap is in closed position (handle facing up) - only applicable to bags with tap.
- Attach bag onto leg, either above or below knee using a strap or securement device with catheter connector up (you may cut the strap to fit your size).
- Remove cap from entry port and connect port directly to catheter or tubing.
- To empty bag, hold the bottom of the bag steady, flip the Flip-Eze tap open and drain carefully - only applicable to bags with tap.



### PRODUCT INFORMATION

Material	PVC
Tube Length	50cm
Sterility	Sterile & Non-Sterile available
Shelf Life	5 Years
Latex Status	Latex Free

### ORDERING DETAILS

STYLE	Sterile	Sterile	Non-Sterile	Non-Sterile
SIZE	500ml	750ml	500ml	750ml
CH2 CODE	2556723	2556731	2556740	2556758
UNIT OF MEASURE	Each	Each	Pack of 10	Pack of 10