

baremedical®

Urine Drainage Bags

Baremedical urine drainage bags can measure up to 2000ml and are suitable for overnight use. Both the sterile and non-sterile bags have universally spaced holes above the bag to provide easy hanging ability. Our sterile drainage bags contains an easily accessible needle-less sample port and t-valve.

Features & Benefits

- All-in-one drainage system for convenient application
- Durable construction for odor containment and leak resistance
- Anti-reflux chamber reduces gross reflux
- Features an easily accessible needle-less sample port on our sterile bags and cap for convenience and ease of use
- Design allows for even filling and drainage
- Perforated tear line on the non-sterile packs for ease of opening and accessing the loose bags inside.

Instructions For Use

- Ensure that the t-valve is in closed position (pushed as far right as possible) - only applicable to bags with tap.
- Attach bag onto side of bed or any functional place that will not be in the way of staff or patient.
- Remove cap from entry port and connect port directly to catheter or tubing.
- To empty bag, hold the bottom of the bag steady, push the t-valve to the open position (as far left as possible) and drain carefully - only applicable to bags with tap.



PRODUCT INFORMATION

Material	PVC
Tube Length	Sterile: 120cm, Non-Sterile: 100cm
Sterility	Sterile & Non-Sterile available
Shelf Life	5 Years
Latex Status	Latex Free

ORDERING DETAILS

SIZE	2000ml Sterile	2000ml Non-Sterile
CH2 CODE	2556766	2556774
UNIT OF MEASURE	Each	Pack of 10