

baremedical®

Catheters Hydrophilic

Baremedical hydrophilic catheters are pre-lubricated and hydrated to help promote excellent gliding properties. With easier, more comfortable insertion, hydrophilic catheters offer decreased friction. This leads to better patient satisfaction by helping reduce the risk of urethral damage.

Features & Benefits

- Staggered eyes for maximum urine flow
- Polished eyes to help minimize urethral trauma
- Flexible funnel end with ridged design for grip and ease of handling
- Color-coded funnel and packaging label for easy identification of French size
- Packaging design includes loop cutout for hanging to maintain sterility
- Individually packaged for single, sterile use
- Not made with DEHP, DINP, BPA or natural rubber latex
- Length - Male 40cm, Female 20cm

Instructions For Use

- Infiltrate the catheter in sterile water for 30s.
- Open the single package, take the connector end by hand and take out the catheter.
- Carefully insert the product from the urethral orifice (touch free can be used during insertion). and urine will flow out when the lateral orifice of the catheter is inserted into the body.
- Begin to drain.
- After catheterization, gently pull out the catheter, if met with resistance; Please slowly rotate it as it is withdrawn and discard it safely.



PRODUCT INFORMATION

Material	PVC
Sterility	Sterile
Shelf Life	3 years
Latex Status	Latex Free

ORDERING DETAILS

SIZE	Female 10	Female 12	Female 14	Male 10	Male 12	Male 14	Male 16
CH2 CODE	2564310	2564328	2564336	2564344	2564352	2564361	2564379
UNIT OF MEASURE	Each	Each	Each	Each	Each	Each	Each