

baremedical[®]

Catheters Nelaton

Baremedical intermittent catheters help minimize urethral trauma and reduce the chance of bringing bacteria into the bladder. These PVC uncoated catheters are ready-to-use and designed to help your patients self-catheterize quickly and easily.

Features & Benefits

- Staggered eyes for maximum urine flow
- Polished eyes to help minimize urethral trauma
- Flexible funnel end with ridged design for grip and ease of handling
- Color-coded funnel and packaging label for easy identification of French size
- Packaging design includes loop cut out for hanging to maintain sterility
- Individually packaged for single, sterile use
- Not made with DEHP, DINP, BPA or natural rubber latex
- Length - Male 40cm, Female 17.5cm

Instructions For Use

- Take out the product after the package is opened, apply lubricant on the tube, and pay attention to prevent the connector from being polluted.
- Carefully insert the product from the urethral orifice (touch free can be used during insertion). and urine will flow out when the lateral orifice of the catheter is inserted into the body.
- Begin to drain.
- After catheterization, gently pull out the catheter, if met with resistance; Please slowly rotate it as it is withdrawn and discard it safely.



PRODUCT INFORMATION

Material	PVC
Sterility	Sterile
Shelf Life	5 years
Latex Status	Latex Free

ORDERING DETAILS

SIZE	Female 10	Female 12	Female 14	Male 10	Male 12	Male 14	Male 16	Male 18
CH2 CODE	2564395	2564408	2564416	2564424	2564432	2564441	2564459	2564820
UNIT OF MEASURE	Each	Each	Each	Each	Each	Each	Each	Each