baremedical[®] Catheters Ready To Use

Baremedical ready to use catheters are pre-soaked, making it easier and more comfortable to catheterize whilst minimizing the risk of urethral damage. No additional lubricant or sterile water additive required. Supplied in sterile sealed packaging with the catheter being immersed in saline solution.

Features & Benefits

- Staggered eyes for maximum urine flow
- Polished eyes to help minimize urethral trauma
- Flexible funnel end with ridged design for grip and ease of handling
- Color-coded funnel and packaging label for easy identification of French size
- Packaging design includes loop cutout for hanging to maintain sterility
- Individually packaged for single, sterile use
- Not made with DEHP, DINP, BPA or natural rubber latex
- Length Male 40cm, Female 20cm

Instructions For Use

- Peal open the packet and remove the catheter by grasping the funnel. When doing so, take care not to contaminate the catheter.
- Using the funnel to help position the catheter with one hand, use the other hand to insert the lubricated catheter gently
- Begin to drain.
- After catheterization, gently pull out the catheter, if met with resistance; Please slowly rotate it as it is withdrawn and discard it safely.

	10
baremedical Internation	A ANTINATION AND A ANTINATION ANTIN'A
Acception 1000 Boold of the sector 10000 Boold of the sect	A Construction of the second s
	Transfer A

PRODUCT INFORMATION				
Material	TPU			
Sterility	Sterile			
Shelf Life	2.5 years			
Latex Status	Latex Free			

ORDERING DETAILS

SIZE	Female 10	Female 12	Female 14	Male 10	Male 12	Male 14	Male 16	Male 18		
CH2 CODE	2564740	2564758	2564766	2564774	2564782	2564791	2564803	2564811		
UNIT OF MEASURE	Each	Each	Each	Each	Each	Each	Each	Each		

